

INSTRUCTIONS FOR USE: THE PINK HIP KIT®

Product 40618STRBL

1. Place The Pink Hip™ Pad at the perineal section of the table so that the pad makes direct contact with the table's surface. Position the white Velcro straps on the underside of The Pink Hip Pad so that the "This Side Up" tag is facing upwards.
2. Attach both white Velcro straps to the surgical bed rails by looping under the rail and affixing the ends of the Velcro together.
3. Attach purple strap under the surgical table by first placing the longer end of the purple straps under the table, followed by overlapping the longer end of the strap with the shorter end of the strap. Make sure to press the white hook material of the shorter end of the strap firmly onto the longer end of the strap to secure in place.
4. Place head positioner above The Pink Hip Pad in a location sufficient for the patient head placement.
5. Lay the lift sheet over the pad, centered between the purple and white straps. The lift sheet should cover only the portion of the pad that will be addressing the small of the patient's back – below the Scapula Region and above the Sacrum. Begin IV or intubation according to hospital protocol.
6. Begin positioning the patient on The Pink Hip Pad with the lift sheet provided. Utilize the lift sheet to carefully lift the patient up off the pad to reposition as needed for safe and proper application. Do not drag patient on pad.
7. Place one Arm Board Pad under the patient's arm that is the opposite side of the operative hip. Place arm on arm board with palms up, or per hospital positioning protocol.
8. Put patient's feet into the traction boot liners and wrap the feet per hospital protocol.
9. Lay other arm across patient's chest with the other arm board pad so that it cushions the patient's arm on their chest.
10. Place purple body straps across patient's arm and attach the body straps as follows:
 - 10 A. Place the strap component with the small Velcro square around the table's accessory rail and through strap buckle
 - 10 B. Repeat the above step on the other side of the table with the remaining strap component with the Velcro's hook side facing downward. Join the straps together so that it goes over the patient's arm and chest
11. Place the patient's wrapped feet into the stirrups and begin postless hip distraction process

IMPORTANT:

- Position patient so that the shoulders do not extend past the top edge of the pad.
- Positioned the patient with approximately six (6) inches of material between the patient's gluteal crease and the distal edge of the pad.
- If patient presents an anatomical challenge during attempted distraction, the use of Air Arthrogram could ease the distraction process



ATTENTION:

- A. Before using The Pink Hip Kit, ensure O.R. table pad is securely affixed to the O.R. table.
- B. Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Check for skin integrity and proper circulation. Product is to be used by a licensed medical professional only.

LITIFUPP016 R1 5/27/2021



702 Prominence Drive
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500
Fax: 724.337.0555



xodusmedical.com
info@xodusmedical.com

FDA REGISTERED
ISO 13485 CERTIFIED


xodusmedical.com/eifu

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands