


# PinkProtect ICU Positioning Kit Instructions for Use:

1. Remove individual positioners from packaging. To ensure full expansion of positioners, wait at least five minutes prior to positioning patients.
2. Individual PinkProtect positioners are designed to protect bony prominences, patient areas prone to Hospital Acquired Pressure Injury, etc. They may be adjusted, aligned, and positioned based on patient anatomy, hospital protocol, etc. The following serves as a general guideline:
  - A. Place the SideProne headrest on the ICU bed relative to the anticipated position of the patient's head. Align the ET Tube channel with the patient's airway, allowing the intubation tube to be routed, kink-free, away from the patient's head when positioned on its side.
  - B. Place the included Pink Arcs beneath any combination of heels, tops of feet, knees, neck, chest and torso depending on procedural requirements, patient anatomy, etc.
  - C. Align Dual Layer Pads for general skin protection and pressure distribution under knees, thighs, elbows, chest, etc.

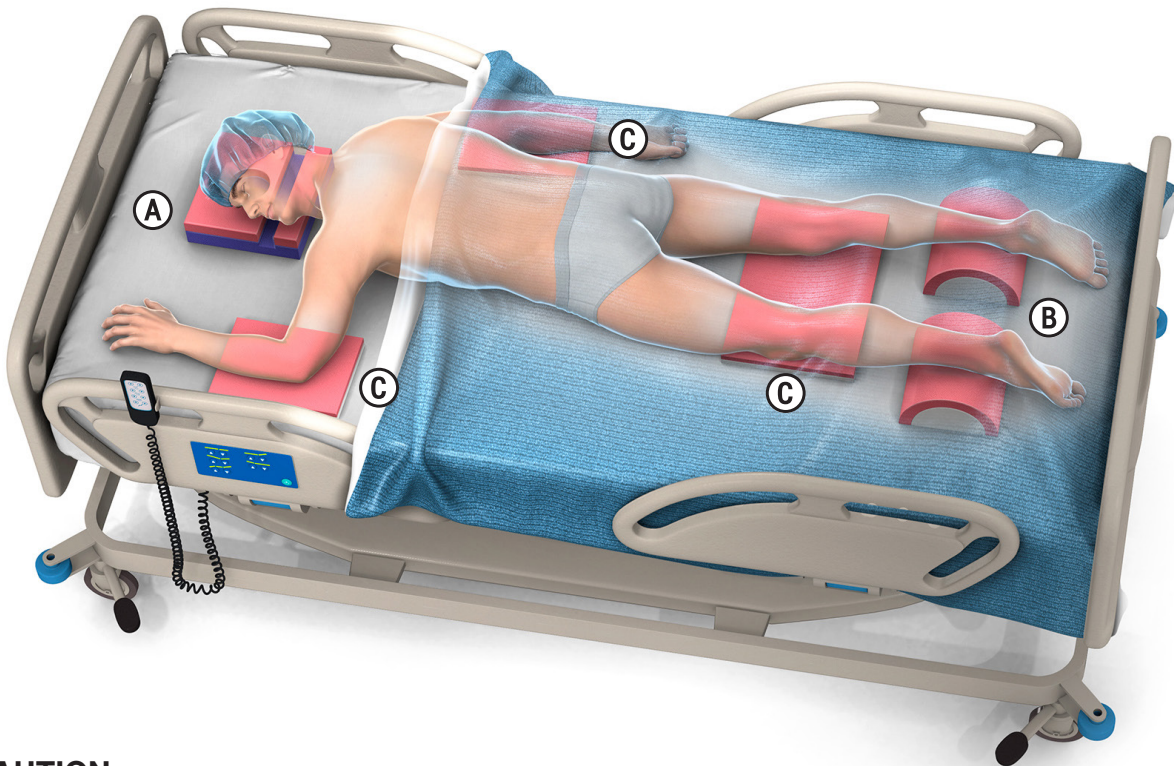

## PRECAUTION:

The contents of this kit are intended for SINGLE-USE only, do not re-use.

## WARNINGS:

Frequently monitor the patient's neck, head, eyes, nose and mouth to ensure that a safe position is maintained and that the ET tube has not been kinked or displaced. Failure to do so can lead to serious adverse consequences including blindness and/or failure to ventilate. Frequently monitor sensitive skin areas and readjust patient when necessary to avoid risk of pressure injury.



702 Prominence Drive  
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500  
Fax: 724.337.0555



xodusmedical.com  
info@xodusmedical.com



FDA REGISTERED  
ISO 13485 CERTIFIED



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

LITIFUIC001 R0 7/21/2020