

One-Step™ Face Protector

1. Place the Headrest

Position the headrest as shown, with the top of the headrest closest to the anesthesia end of the table. The top of the headrest features the longer of two belts labeled forehead strap. (Figure 1)

2. Position the Patient

Position the intubated patient, centering the patient's head on the headrest. (Figure 1)

3. Place the One-Step Face Protector

Place the Face Protector over the patient's face as shown. Begin by lifting the velcro® tab on the chin guards of the Face Protector. Spread the chin portion at the dividing split, and pass the intubation tubing through the slit as shown. Place the Face Protector on the patient's face. Then replace the velcro closure. Insets of The Pink Pad® material on the inside of the Face Protector will rest on the patient's forehead, cheeks and chin. (Figure 2)

4. Affix the Head Straps

Affix the first portion of the purple forehead strap (A) located on the right of the patient's face to the velcro of the Face Protector. Secure the remaining half of the forehead strap (B). The intubation tubing may be secured between the forehead straps. (Figure 3)

5. Affix the Chin Straps

Affix the chin straps to secure the Face Protector in place. The set up is now complete. Note that the eye protection visor will permit access by detaching the velcro closure of the visor (located on the patient's right side), allowing the visor to be peeled to the side. (Figure 4)

Figure 1

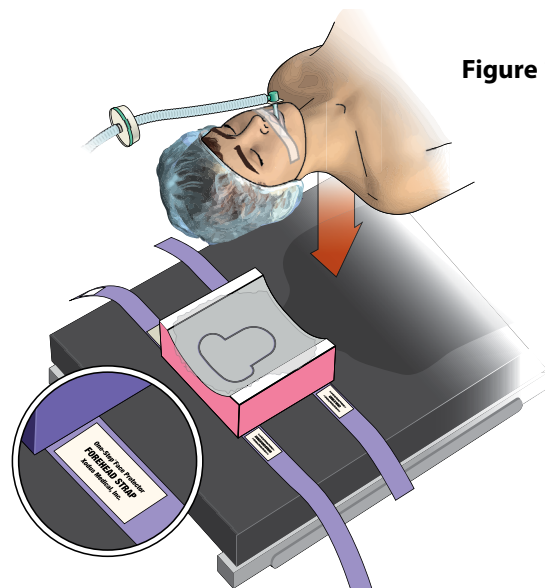


Figure 2

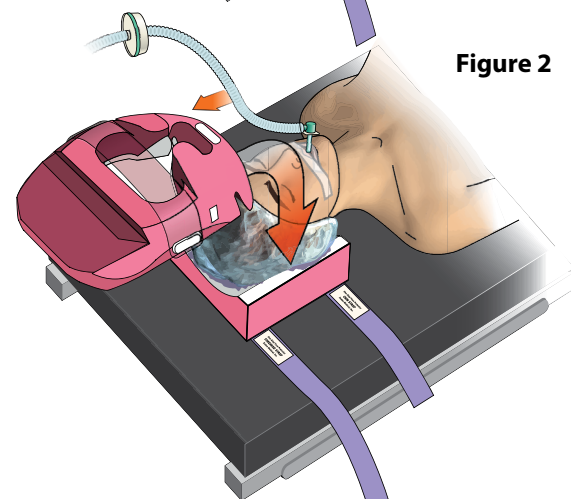


Figure 3


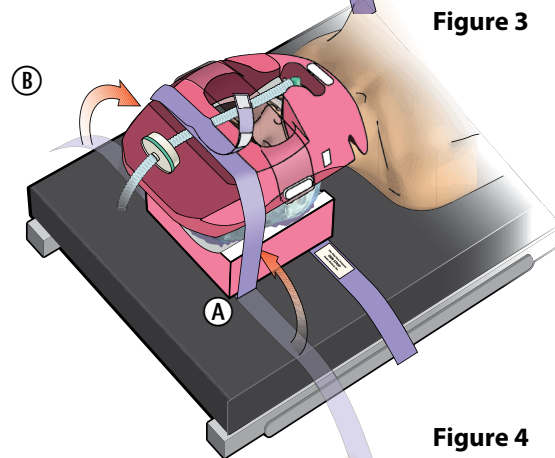


Figure 4

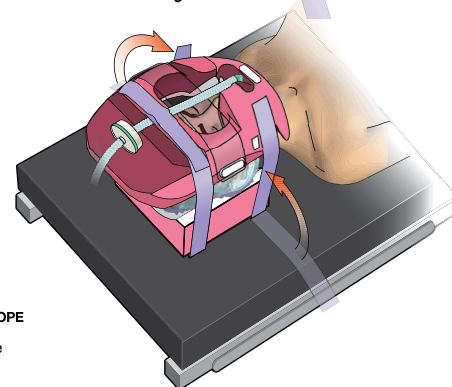

702 Prominence Drive Tel: 724.337.5500
Pittsburgh, PA 15068 U.S.A. Fax: 724.337.0555

xodusmedical.com • info@xodusmedical.com

Patent No. 10,842,206; 10,842,574



FDA REGISTERED
ISO 13485 CERTIFIED



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands