

Prone Head Rest Instructions for Use:


1. Remove prone head rest from packaging. To ensure full expansion of the head rest, wait at least five minutes prior to positioning patients.
2. Place the head rest on the OR table relative to the anticipated position of the patient's head. The "T" shaped hole should be positioned as shown in Figure 1.
3. Place the intubated patient's face onto the head rest and feed the intubation tube through the slot on the side of the head rest. The patient's eyes, eyebrows and nose should be centered within the "T" shaped hole on the head rest to prevent pressure on the patient's eyes.
4. Upon completion of the surgical procedure, remove the intubation tube from the head rest slot BEFORE removing the patient's face from the head rest.

PRECAUTION:

1. This head rest is intended for SINGLE-USE only, do not re-use.

WARNINGS:

1. Frequently monitor the patient's neck, head, eyes, nose and mouth to ensure that a safe position is maintained and that the ET tube has not been kinked or displaced. Failure to do so can lead to serious adverse consequences including blindness and failure to ventilate.

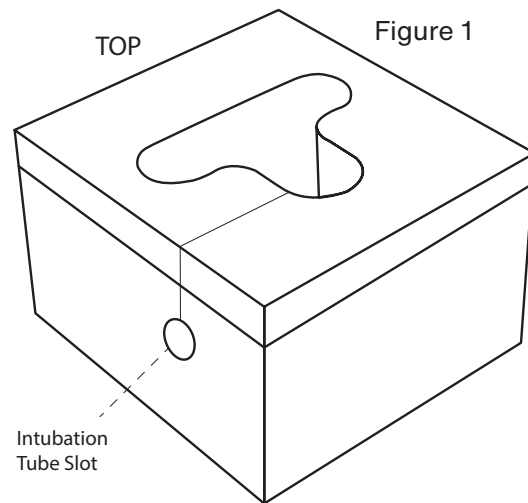

LITIFUPH001 R0 11/1/19



702 Prominence Drive
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500
Fax: 724.337.0555



xodusmedical.com
info@xodusmedical.com

MADE IN THE
USA



FDA REGISTERED
ISO 13485 CERTIFIED

EC	REP
----	-----

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands