


Instructions for Use: Hip Arthroscopy Bootliners - #40532SN


1. Unfold bootliner and have the white DermaProx™ layer facing upwards.

2. Place patient's heel in the middle of the pad so that the notch aligns with patient's calf.

3 AB.

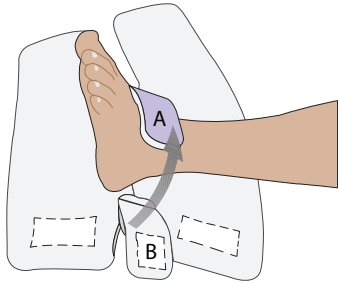


3C.

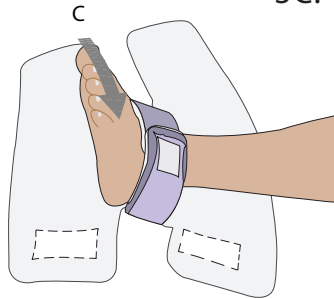


3A. Place strap A (non-velcro side) of middle strap on top of patient's ankle

3B. Place strap B (velcro side) of middle strap on top of strap A so the patient's foot is wrapped. Be sure not to overtighten.

3D.

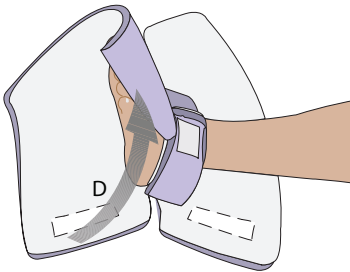


4A.

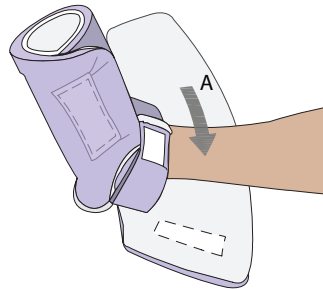


3C. Wrap non-velcro side of the pad around the patient's toes.

3D. Wrap bottom half of pad around the rest of the patients toes and place the velcro on top of the non-velcro side of the pad so that the top of the patient's foot is wrapped. Be sure not to overtighten.

4B.

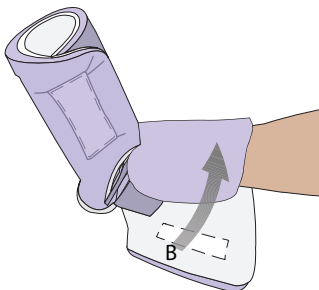

Final Product

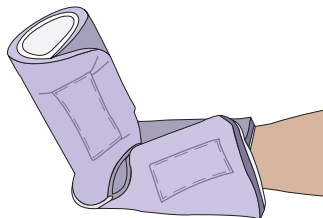

4A. Wrap non-velcro side of the pad around the patient's calf

4B. Wrap bottom half of pad around the rest of the patients calf and place the velcro on top of the non-velcro side of the pad so that the patient's foot is wrapped. Be sure not to overtighten.




702 Prominence Drive
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500
Fax: 724.337.0555

xodusmedical.com | info@xodusmedical.com



MADE IN THE
USA

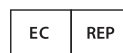

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

FDA REGISTERED
ISO 13485 CERTIFIED

LITIFUPP012 R0 5/2/19