

The Pink Pad[®] XL

Advanced Trendelenburg Positioning System

The Pink Pad XL advanced patient positioning system is the innovative, clinically-preferred product for increasing patient safety in Trendelenburg – preventing unwanted patient movement, while protecting skin and nerves from injury and pressure-related complications.

This second-generation Pink Pad offers greater surface area and features a perineal cutout for better access to the surgical site.

U.S. Patents
8,464,720; 8,511,314; 9,161,876
9,750,656; 9,782,287; 9,931,262
9,949,883; 10,045,902; 10,285,890

European Patent
EP 2 802 305 B1

Additional
Patents
Pending

40595

The Pink Pad XL & One-Step[™]
Trendelenburg Arm Protectors,
& Trendelenburg Head Positioner

- The Pink Pad XL
- Lift sheet
- Set of body straps
- Pair of One-Step[™] Trendelenburg Arm Protectors
- Trendelenburg Head Positioner

40583

The Pink Pad XL

- The Pink Pad XL
- Lift sheet
- Set of body straps

latex
free

**Packaged
Together for
Efficiency
and Savings**

 **XODUS MEDICAL**
Making Surgery Safer[™]


Contact us
for a
FREE sample

Pittsburgh, PA USA • Tel: 724-337-5500 • Fax: 724-337-0555
xodusmedical.com • info@xodusmedical.com

 ISO 13485 CERTIFIED
FDA REGISTERED

 latex
free

**MADE IN THE
USA**

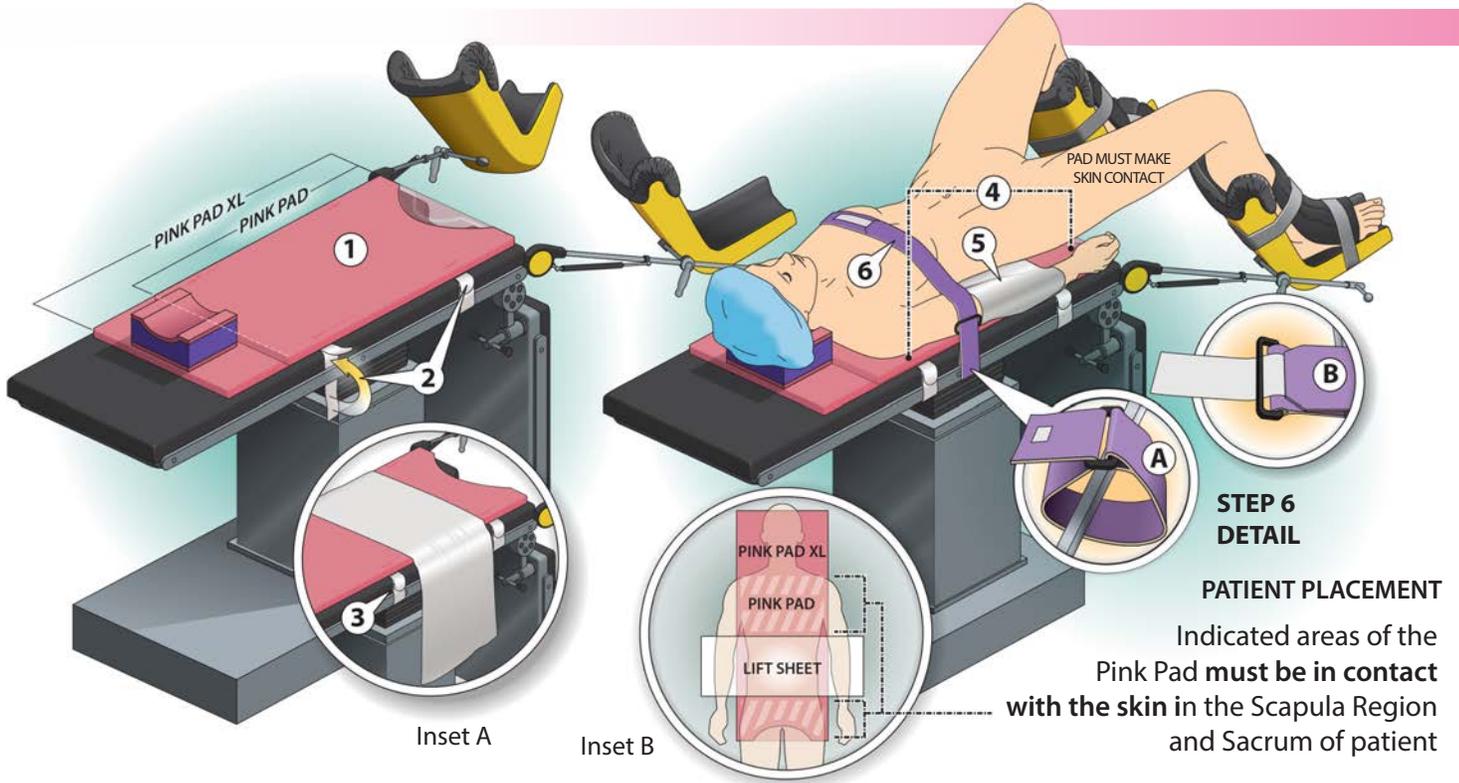

STEP 6 DETAIL

PATIENT PLACEMENT

Indicated areas of the Pink Pad **must be in contact with the skin** in the Scapula Region and Sacrum of patient

1. Place The Pink Pad at the very edge of the middle table segment nearest the perineal cutout, positioning the white Velcro straps on the underside of The Pink Pad in direct contact with the table pads surface. **DO NOT** use in conjunction with table covers or other cloth. The Pink Pad must contact the surface of the table pad. The "This Side Up" tags should be facing upward.
2. Attach the white Velcro straps of The Pink Pad to the surgical bed rails by looping under the rail, as shown, and affixing the ends of the Velcro to each other.
3. Lay the lift sheet over the pad, centered between the Velcro straps as shown (inset A). The lift sheet should cover only the portion of the pad that will be addressing the small of the patient's back - below the Scapula Region and above the Sacrum.
4. Follow hospital protocol for intubation. Then properly position patient on pad.
IMPORTANT - THE PATIENT'S SKIN MUST MAKE DIRECT CONTACT WITH THE PINK PAD. The skin of the Scapula Region and Sacrum must contact the pad surface (inset B). Utilize the included lift sheet to carefully lift the patient up and off the pad to reposition as needed for safe and proper application of stirrups. Do not drag patient on pad. Make sure pad remains completely flat at all times.
5. Tuck arms as shown or per hospital protocol.
6. Attach the body strap as follows:
 - A. Place the strap component with the small Velcro square around the table's accessory rail and through strap buckle.
 - B. Repeat the above step on the other side of the table with the remaining strap component with the Velcro's hook side facing downward. Join straps together.

ATTENTION:

1. Before using The Pink Pad, ensure that the O.R. table pad is securely affixed to the O.R. table and is clean and free of residue
2. Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Check skin for integrity and proper circulation. Product is to be used by licensed medical professional only.
3. Care should be taken to safeguard The Pink Pad from exposure to prep solutions.
4. Handling & Storage: During ALL handling and storage, assure that the pad is flat. Do not roll or fold the pad.



702 Prominence Drive
Pittsburgh, PA 15068 U.S.A.
Tel: 724.337.5500
Fax: 724.337.0555
xodusmedical.com | info@xodusmedical.com



MADE IN THE
USA

FDA REGISTERED
ISO 13485 CERTIFIED



LITCOMBO002 R10 5/22/19

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

U.S. Patents:

8,464,720; 8,511,314; 9,161,876
9,750,656; 9,782,287; 9,931,262
9,949,883; 10,045,902; 10,285,890

European Patent:

EP 2 802 305 B1
Additional Patents Pending