

# Instructions for Use: Beach Chair Head Restraint – Loop Material Straps

**BEFORE USING PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.**

## DESCRIPTION

The Xodus Medical Beach Chair Head Restraint is a foam based head restraint for use in the Beach Chair position for shoulder arthroscopy surgical procedures.

## INDICATIONS

The Beach Chair Head Restraint is a single use, non-sterile device to be used in surgical procedures.

## INTENDED PURPOSE AND USER

This device is intended for use by trained healthcare professionals only to retain a patients head positioning during shoulder arthroscopy surgical procedures.

## CONTRADICTIONS

This device is not designed, sold, or intended for use except as indicated.

## KNOWLEDGE AND USE

Professional use requires knowledge of this instruction for use. Device use limited to surgical operating room in a hospital or surgery center.


## CLINICAL BENEFITS

Provides head restraint and stability during the Beach Chair shoulder arthroscopy surgical procedure.


## PREPARATION AND USE

1. Apply the Beach Chair Head Restraint to the patient's face with the DermaProx™ (white) layer facing the patient's skin. Center the restraint like a mask around the patient's face, leaving the eyes, nose, and mouth free from obstruction.
  - A. Head Restraint - Inside View (Contacts Patient) and Application to patient's skin.


Step A

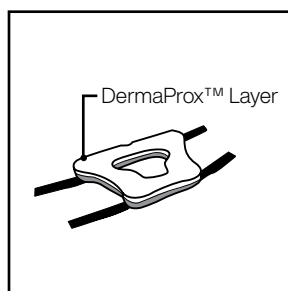


Step B

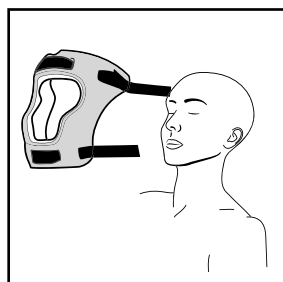

Step C

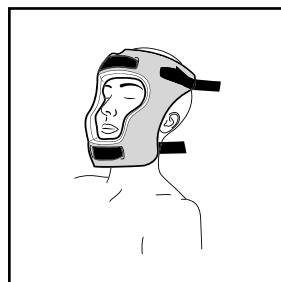

Step D

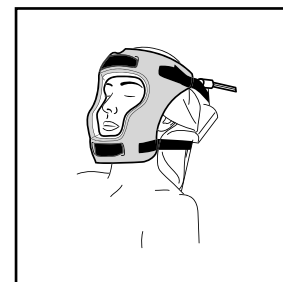

2. Attach each of the four straps on the sides of the restraint directly to the Beach Chair frame, safely securing the patient's head in place.
3. Adjust straps to provide secure positioning. **WARNING:** Avoid over-tightening.

## DISPOSAL

- After use, the Beach Chair Head Restraint should be disposed of in accordance with hospital policy.

## WARNINGS

- Do not over-tighten the Beach Chair Head restraint.
- This device was designed, test and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure or subsequent injury.
- Reprocessing of this device may create the risk of contamination and patient infection.
- Do not reuse or reprocess this device.
- After use, the Beach Chair Head Restraint should be disposed of in accordance with hospital policy.
- The Beach Chair Head Restraint should be stored in a clean, dry location at room temperature prior to use. Avoid prolonged exposure to elevated temperatures.
- Please visually inspect for breaches of packaging integrity prior to use. Do not use if damaged, opened or breached.
- USA Federal law restricts this device to sale by or on the order of a physician.

## PRECAUTIONS

- The Beach Chair Head Restraint should be used in accordance with the instructions for use and any contraindications, warnings or precautions provided by the manufacturer of the associated instrument.
- Notice to the User and/or patient that any serious incident that has occurred in relation to the device should be reported and the competent authority of the Member State in which the user and/or patient is established, as well as, Xodus Medical and its Authorized Representative.

## TECHNICAL SPECIFICATIONS

- Materials of manufacture include:
  - Polyurethane Foam
  - Nylon
- Shelf Life – Indefinite

## STORAGE, TRANSPORT, AND OPERATIONAL CONDITIONS

- Beach Chair Head Restraint should be stored in a clean, dry location at room temperature prior to use. Avoid prolonged exposure to elevated temperatures.



Caution, Consult  
Accompanying Documents



Medical Device



Consult instructions for  
use or consult  
electronic instructions  
for use




Do Not Re-use



Not Made with Natural  
Rubber Latex



Do Not Use If The  
Package Is Damaged  
Or Opened

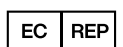

Emergo Europe  
Westervoortsedijk 60  
6827 AT Amhem  
The Netherlands



702 Prominence Drive Ph. 1-724.337.5500  
Pittsburgh, PA 15068 U.S.A.

FDA REGISTERED  
ISO 13485 CERTIFIED

xodusmedical.com | info@xodusmedical.com

LITIFUBC003 R1 5/30/2023