

# INSTRUCTIONS FOR USE: THE PINK ARCS

## BEFORE USING PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.

### DESCRIPTION

The Pink Arcs is a single use device used in surgical procedures. The intended use of the Pink Arcs is to provide protection, proper surgical positioning, and cushioning when positioning patients for surgical positioning and pressure related injury prevention.

### INDICATIONS

The Pink Arcs is a single use, non-sterile device to be used during surgery to provide protection, proper surgical positioning, and cushioning in the prevention of patient injury.

### INTENDED PURPOSE AND USER

This device is intended for use by trained healthcare professionals only.

### CONTRADICTIONS

This device is not designed, sold, or intended for use except as indicated.

### KNOWLEDGE AND USE




Professional use requires knowledge of this instruction for use. Device use limited to surgical operating room in a hospital or surgery center.

### CLINICAL BENEFITS

Provides protection, proper surgical positioning, and cushioning in the prevention of patient injury during surgical procedures.

### PREPARATION AND USE

- Pink Arcs can be used for a variety of positioning needs rendered by surgical need.
- Place Pink Arcs in direct contact with skin to utilize the unique softness, immersion, and envelopment characteristics to help protect against positional injuries.
- Make sure to continuously check patient skin integrity and proper circulation.
- Choose from a variety of Pink Arc sizes to deliver optimal pressure distribution specific to surgical procedure and patient population.
- Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.
- Product is to be used by licensed medical professionals only.
- Care should be taken to safeguard the Pink Arcs from exposure to prep. solutions.

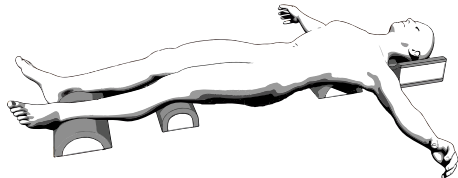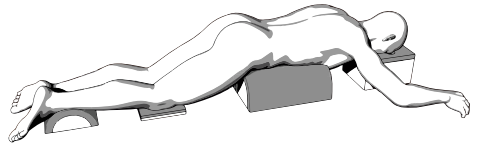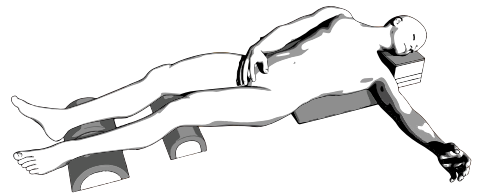

## DISPOSAL

- After use, the Pink Arcs should be disposed of in accordance with hospital policy.

## WARNINGS

- This device was designed, test and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure or subsequent injury.
- Reprocessing of this device may create the risk of contamination and patient infection.
- Do not reuse or reprocess this device.
- After use, the Pink Arcs should be disposed of in accordance with hospital policy.
- Check skin for integrity and proper circulation.
- The Pink Arcs should be stored in a clean, dry location at room temperature prior to use. Avoid prolonged exposure to elevated temperatures.

## PRECAUTIONS

- The Pink Arcs is for use in perioperative settings only.
- The Pink Arcs should be used in accordance with the instructions for use and any contraindications, warnings or precautions provided by the manufacturer of the associated instrument.
- Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.
- Notice to the User and/or patient that any serious incident that has occurred in relation to the device should be reported and the competent authority of the Member State in which the user and/or patient is established, as well as, Xodus Medical and its Authorized Representative.

## TECHNICAL SPECIFICATIONS

- Materials of manufacture include:
  - » Polyurethane foam
- Shelf Life – indefinite

## STORAGE, TRANSPORT, AND OPERATIONAL CONDITIONS

- The Pink Arcs should be stored in a clean, dry location at room temperature prior to use. Avoid prolonged exposure to elevated temperatures.



702 Prominence Drive  
New Kensington, PA 15068,  
United States

xodusmedical.com  
info@xodusmedical.com  
Tel: 1+724-337-5500

FDA REGISTERED  
ISO 13485 CERTIFIED



LITIFUPA001 R1 1/4/2024