

# INSTRUCTIONS FOR USE: PRONE HEAD POSITIONER


1. Remove prone head positioner from packaging. To ensure full expansion of the head positioner, wait at least five minutes prior to positioning patients.
2. Place the head positioner on the OR table relative to the anticipated position of the patient's head. The "T" shaped hole should be positioned as shown in Figure 1.
3. Place the intubated patient's face onto the head positioner and feed the intubation tube through the slot on the side of the head positioner. The patient's eyes, eyebrows and nose should be centered within the "T" shaped hole on the head positioner to prevent pressure on the patient's eyes.
4. If needed, the overall height of the head positioner may be adjusted lower by 1.5". This adjustment is done as follows:
  - Firmly hold the top of the head positioner and the bottom section of the head positioner (at the perforation)
  - Slowly pull the bottom perforation away from the top of the head positioner.
  - Be sure the top of the head positioner is separating from the bottom only at the perforation on each side of the "T" shape
5. Upon completion of the surgical procedure, remove the intubation tube from the head positioner slot BEFORE removing the patient's face from the head positioner.

## PRECAUTION:

1. This headrest is intended for SINGLE-USE only, do not re-use.

## WARNINGS:

1. Frequently monitor the patient's neck, head, eyes, nose and mouth to ensure that a safe position is maintained and that the ET tube has not been kinked or displaced. Failure to do so can lead to serious adverse


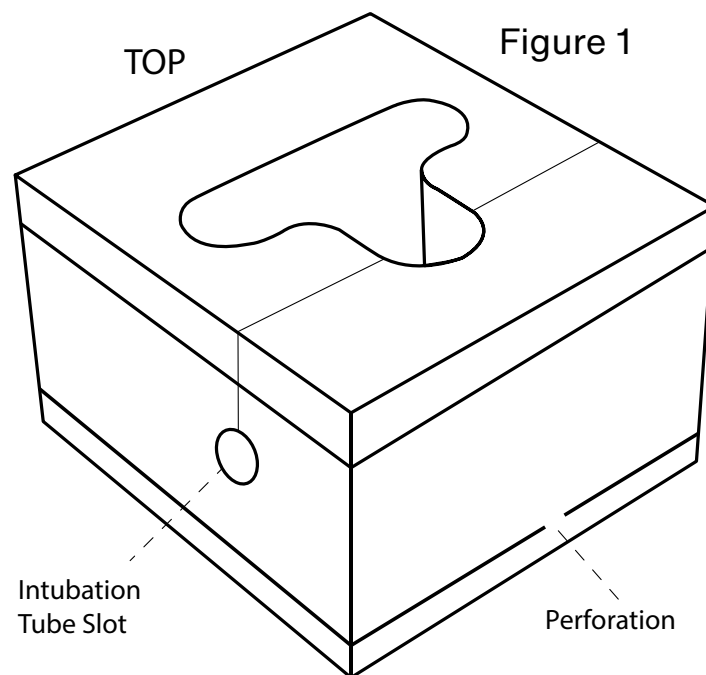


LITIFUPH002 R0 1/9/2023



702 Prominence Drive  
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500  
Fax: 724.337.0555

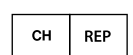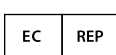

702 Prominence Drive  
Pittsburgh, PA 15068 U.S.A.

xodusmedical.com  
info@xodusmedical.com

FDA REGISTERED  
ISO 13485 CERTIFIED



xodusmedical.com/eifu

EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

MedEnvoy  
Switzerland  
Gotthardstrasse  
28 6302 Zug  
Switzerland