

# INSTRUCTIONS FOR USE: PINKPROTECT CHILD PRONE HEAD POSITIONER—XS

**BEFORE USING PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.**

## DESCRIPTION

The PinkProtect Child Prone Head Positioner—XS is a single use device used in surgical procedures. The intended use of the PinkProtect Child Prone Head Positioner—XS is to provide protection, proper surgical positioning, and cushioning when positioning patients for surgical positioning and pressure related injury prevention.

## INDICATIONS

The PinkProtect Child Prone Head Positioner—XS is a single use, non-sterile device to be used during surgery to provide protection, proper surgical positioning, and cushioning in the prevention of patient injury.

## INTENDED PURPOSE AND USER

This device is intended for use by trained healthcare professionals only.

## CONTRADICTIONS

This device is not designed, sold, or intended for use except as indicated.

## KNOWLEDGE AND USE


Professional use requires knowledge of this instruction for use. Device use limited to surgical operating room in a hospital or surgery center.

## CLINICAL BENEFITS

Provides provide protection, proper surgical positioning, and cushioning in the prevention of patient injury during surgical procedures.

## PREPARATION AND USE

1. Remove PinkProtect Child Prone Head Positioner—XS from packaging. To ensure full expansion of the headrest, wait at least five minutes prior to positioning patients.
2. Place the PinkProtect Child Prone Head Positioner—XS on the OR table relative to the anticipated position of the patient's head. The "T" shaped hole should be positioned as shown in Figure 1.
3. Place the intubated patient's face onto the headrest and feed the intubation tube through the slot on the side of the headrest. The patient's eyes, eyebrows and nose should be centered within the "T" shaped hole on the head rest to prevent pressure on the patient's eyes.
4. Upon completion of the surgical procedure, remove the intubation tube from the PinkProtect Child Prone Head Positioner—XS slot BEFORE removing the patient's face from the headrest.

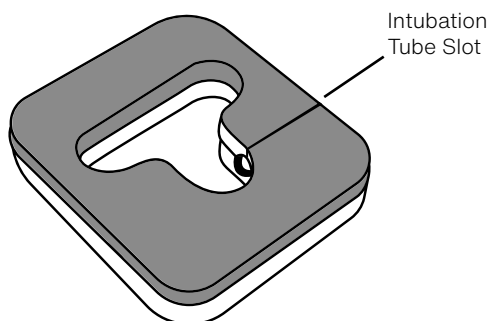

## DISPOSAL

- After use, the PinkProtect Child Prone Head Positioner—XS should be disposed of in accordance with hospital policy.

## WARNINGS

- This device was designed, test and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure or subsequent injury.
- Reprocessing of this device may create the risk of contamination and patient infection.
- Do not reuse or reprocess this device.
- After use, the PinkProtect Child Prone Head Positioner—XS should be disposed of in accordance with hospital policy.
- The PinkProtect Child Prone Head Positioner—XS should be stored in a clean, dry location at room temperature prior to use. Avoid prolonged exposure to elevated temperatures.

## PRECAUTIONS

- The PinkProtect Child Prone Head Positioner – XS is for use in perioperative settings only.
- The PinkProtect Child Prone Head Positioner—XS should be used in accordance with the instructions for use and any contraindications, warnings or precautions provided by the manufacturer of the associated instrument.
- Frequently monitor the patient's neck, head, eyes, nose and mouth to ensure that a safe position is maintained and that the ET tube has not been kinked or displaced. Failure to do so can lead to serious adverse consequences including blindness and failure to ventilate.
- Notice to the User and/or patient that any serious incident that has occurred in relation to the device should be reported and the competent authority of the Member State in which the user and/or patient is established, as well as, Xodus Medical and its Authorized Representative.

## TECHNICAL SPECIFICATIONS

- Materials of manufacture include:
  - » Polyurethane foam
- Shelf Life – indefinite

## STORAGE, TRANSPORT, AND OPERATIONAL CONDITIONS

- PinkProtect Child Prone Head Positioner—XS should be stored in a clean, dry location at room temperature prior to use. Avoid prolonged exposure to elevated temperatures.



702 Prominence Drive    Ph. 1-724.337.5500  
Pittsburgh, PA 15068 U.S.A.

FDA REGISTERED  
ISO 13485 CERTIFIED

xodusmedical.com | info@xodusmedical.com

LITIFUPS004 R0 6/7/2023