

INSTRUCTIONS FOR USE: CAMEL MAT DISPOSAL WITH SOFT STEP


CAMEL MAT with SOFT STEP

1. Position Soft Step on the surface at its designated location.
2. Unfurl the Camel Mat to its full length and width.
3. Position the Camel Mat over the Soft Step mat such that it is centered over the Soft Step mat. Place the Camel Mat.
4. Assure that the Camel Mat edges are in contact with the surface extending beyond the Soft Step on all sides. There shall be an approximate 2” overlap of the Camel Mat to the surface on all sides.
5. Make sure Camel Mat is flat with no bumps or irregularities in its positioning to the Soft Step Mat across its surface and to the surface on all sides.
6. The Camel Mat is single use, so following its use, disposal is required.
7. For proper preparation for disposal, refer to Instructions for Use LITIFUCM001.



702 Prominence Drive
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500
Fax: 724.337.0555

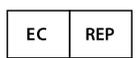

xodusmedical.com
info@xodusmedical.com

FDA REGISTERED
ISO 13485 CERTIFIED


xodusmedical.com/eifu

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

LITIFUCM002 R1 6/1/2021