
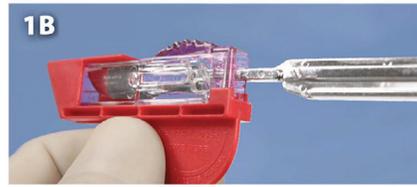


INSTRUCTIONS FOR USE: Safety Scapel

Blade Attachment


1A-B. Grasp blade cartridge by tab of safety cover. With ruler side of handle facing you, insert handle into blade cartridge between center slots.

2A-B. While holding tab, lift cartridge up and out of safety cover to free attached cartridge.

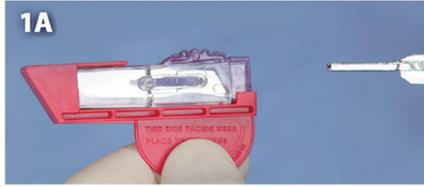


Exposing Blade

1. Depress top button of cartridge shield.

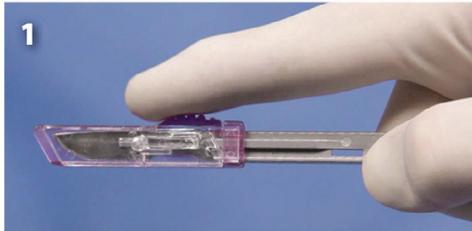


2. Slide back until it locks for full blade exposure.

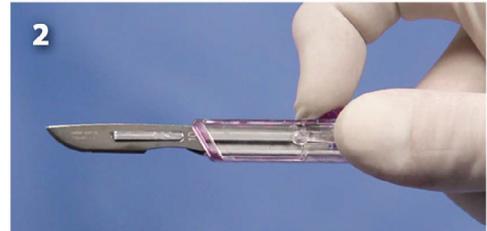


Covering Blade

1. Depress top button of cartridge shield.

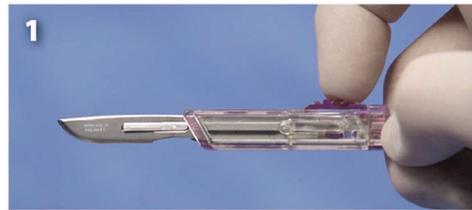

2. Slide cover forward until it locks to cover blade.

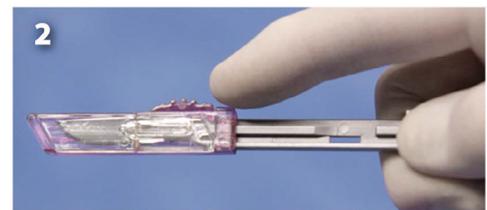


Ejecting Blade Cartridge

1. Depress release button on side of cartridge shield.



2. Slide cartridge forward to eject

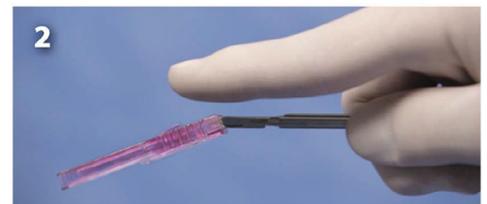

702 Prominence Drive
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500
Fax: 724.337.0555



xodusmedical.com
info@xodusmedical.com

FDA REGISTERED
ISO 13485 CERTIFIED



xodusmedical.com/eifu

LITIFUSS001 R8 6/2/2021